

Thank you for purchasing the PROTON Digital Bathroom Scale PHC 309MS. Please read this instruction manual carefully before using the scale.




### BATTERY

1. Open battery case cover on the back of scale.
2. Load 2xAA Alkaline battery




### OPERATION


**STEP 1.** Put the scale on a hard and flat surface (Avoid carpet or soft surfaces)

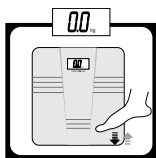

**STEP 2.** Switch the scale on by lightly tapping on the lower right corner of the glass platform. Wait a few seconds until display prompts to "0.0" The scale is now ready to measure your weight.

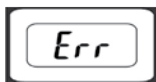

**Note:** The display will show "Err" if you step on the scale before it shows "0.0" In this case, please repeat step 2)

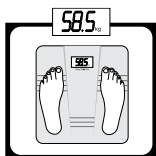


**STEP 3.** Gently step onto the scale. Place your feet evenly on the scale without moving. Numbers on the LCD display will start to increase until the weight is achieved. This display will remain for 8 seconds and then the unit will automatically switch off.



### AUTOMATIC SWITCH OFF


When you step off the scale, it will automatically switch off. Auto shut-off also occurs if display shows "0.0" or shows the same weight reading for approximate 8 seconds.

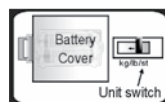

### TO CONVERT WEIGHT UNIT


You can convert weight unit as follows:

1. Find unit switch on the back of scale.
2. Adjust switch to the required setting kg/lb or stone.

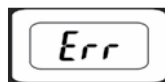


### OVERLOAD INDICATION

Scale overload (Maximum weight capacity is exceeded on the scale). Please step off to avoid damage.

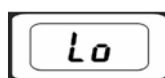

### LOW BATTERY INDICATION

Battery power is low. Please replace battery.

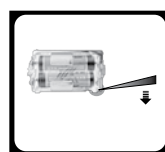

### TO REPLACE BATTERY

1. Open battery case cover on the back of scale.
2. Take out the used battery as shown.
3. Install new battery (2xAA Alkaline) by putting one side of the battery beneath the battery baffle and then pressing down the other side.

### PRECAUTIONS

1. Platform of the scale can be slippery and must be kept clean and dry be careful when stepping on and off the platform.
2. Keep standing still during the course of weighing.
3. DO NOT strike shock or drop the scale.
4. Handle it with care & clean regularly.
5. Always keep your scale in cool dry place.
6. Always keep your scale in horizontal position.
7. If the scale fails to turn on, check whether battery is installed correctly or battery power may be low. Insert or replace battery.
8. If display shows "Err" or scales do not switch off for a long period of time, remove the battery for a few seconds and replace again. If this does not resolve the problem then please contact your dealer for advice.
9. This scale is meant for personal weighing and should not be use for any other purpose.